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Aqua, <0,06% Natriumhypochlorit/
hypochlorige Säure

Lithium-Magnesium-Natrium-Silikat

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**(en) Instructions for use –
Kerrasol™ Wound Gel**

Intended use:

Kerrasol™ Wound Gel is a ready-to-use hydrogel for low-pain application to acute and chronic wounds. It forms a protective film, keeps the wound moist and thus promotes wound healing. Kerrasol Wound Gel is used to cleanse, moisten and keep wounds moist. It forms an ideal wound healing environment and thus supports the natural healing process in the wound. Wound odors are quickly and reliably eliminated using Kerrasol Wound Gel.

Product properties:

The application of Kerrasol Wound Gel is non-irritating, and virtually painless. There is no restriction on the frequency and duration of use. Residues of Kerrasol Wound Gel may remain in the wound. As a multi-dose container, Kerrasol Wound Gel is preserved with NaOCl/HOCl. This allows the use of the opened individual product for 12 weeks.

Application information:

Shake tube well before use!

Before the first application and at dressing changes, it is recommended to use a wound spray or wound irrigation solution to remove exudate, fibrinoid deposits and other contaminants. Suitable for this purpose are products of the line Kerrasol. The frequency of dressing changes depends on the nature of the wound and should be adapted to the particular wound situation. Generously cover the wound surface with Kerrasol Wound Gel and dress the wound for protection. Kerrasol Wound Gel can be used in combination with common compresses, plasters and other common dressing materials. Please also refer to the instructions for use of the respective product. Kerrasol Wound Gel can be used without restriction regarding age and gender of the patient. There are no known undesirable side effects. A slight chlorine odor is a sign of the activity of the preservative system. Kerrasol Wound Gel can be used by non-professionals if they have been instructed by healthcare professionals.

Application restrictions:

Kerrasol Wound Gel should not be used if there are known allergies to any of the ingredients. Do not use in combination with other wound cleaning sprays, solutions, gels and pre-saturated dressings. Do not use simultaneously with products containing chlorhexidine. Do not use Kerrasol Wound

Gel on bleeding or exuding wounds. For use during pregnancy, while breast-feeding and in children under two years of age, consult your doctor or pharmacist beforehand. In children up to 12 years of age, the application should be supervised by an adult. If there is redness, swelling or pain in the wound area or if the condition of the wound does not improve within a few days, a doctor should be consulted.

Safety instructions:

Do not swallow. Avoid contact with eyes. Keep safe from children. Use only tubes that are in perfect condition and undamaged. Use the product only until the stated expiry date and not longer than 12 weeks after opening. Please report a potentially serious incident involving the product immediately to Prontomed (vigilanz@prontomed.de) and the competent authority in your country.

Ingredients:

Aqua, <0,06% sodium hypochlorite/
hypochlorous acid

Lithium magnesium sodium silicate

Symbol Glossary:

	CE Mark
	Keep out of reach of children
	Do not use if package is damaged and consult instructions for use
	Temperature limits
	Keep dry
	Keep away from sunlight
	Manufacturer
	Use-by date
	Batch code
	Catalogue number
	Medical Device
	Unique Device Identifier
	Consult instructions for use or consult electronic instructions for use
	Green Dot

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Distributed by:
3M Deutschland GmbH
Health Care Business
Carl-Schurz-Str. 1
41453 Neuss, Germany
3M.com
For more information see,
HCBGRegulatory.3M.com

3M and Kerrasol are trademarks of 3M. © 2022, 3M. Unauthorized use prohibited. All rights reserved. Made in Germany Issue 02 - Aug 2022 Issue date: 2022-08

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**Prontomed GmbH**