de Gebrauchsinformation - 3M™ Kerrasol™

Wundspüllösung 1000 ml

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Instruction for Use - 3M™ Kerrasol™ Wound Irrigation

Solution 1000 ml

Product properties:

3M™ Kerrasol™ Wound Irrigation Solution is used for cleaning, moistening, mechanical decontamination and infection prevention in the acute and chronic wound.

The application of Kerrasol Wound Irrigation Solution is non-irritating. Kerrasol Wound Irrigation Solution is used for cleaning the wound by helping to remove biofilms and fibrinoid residues. In addition, Kerrasol Wound Irrigation Solution can aid the gentle removal of dressings.

Suitable for use in negative pressure wound therapy (NPWT).

As a multi-dose container, Kerrasol Wound Irrigation Solution is preserved with NaOCl/HOCl. This allows the product to be used for up to 12 weeks after first opening.

Wound odour is quickly and reliably eliminated when using Kerrasol Wound Irrigation Solution.

There are no known side effects, allergic reactions are not to be expected.

Application:

The wound should first be cleaned to remove dirt, blood and any residues. The affected area is thoroughly cleaned using generous amounts of Kerrasol Wound Irrigation Solution. In the event of major contamination, the application can be repeated several times. Once the cleaning procedure is finished, a secondary cleaning solution is not required (i.e. any residual Kerrasol Wound Irrigation Solution can be left in the wound).

Dressings or bandages that are difficult to remove are soaked in Kerrasol Wound Irrigation Solution until they are wet. After a short waiting period, they can then be carefully removed virtually pain-free. If needed, 3M™ Kerrasol™ Wound Gel or 3M™ Kerrasol™ Wound Spray Gel can be applied.

There are no restrictions regarding the frequency and duration of use.

A slight chlorine odour indicates the activity of the preservative system.

Application restrictions:

Kerrasol Wound Irrigation Solution should not be used if there are known allergies to any of the ingredients. Do not use in combination with other wound cleaning sprays, solutions, gels and pre-saturated dressings.

Do not use simultaneously with products containing chlorhexidine.

For use during pregnancy, while breast-feeding and in children under two years of age, consult your doctor or pharmacist beforehand. In children up to 12 years of age, the application should be supervised by an adult. If there is redness, swelling or pain in the wound area or if the condition of the wound does not improve within a few days, a doctor should be consulted.

Not intended for infusion or injection!

General safety instructions:

Only apply to external dermal wounds, do not swallow and avoid eye contact. Keep away from children. Only use intact, undamaged bottles. Only use the products prior to the expiration date indicated on the bottle. Once opened, product can only be used for a maximum of 12 weeks. Please report a serious incident occurring in relation to the device immediately to Prontomed and the responsible regulatory authority.

Ingredients:

Aqua, <0,08% sodium hypochlorite/hypochlorous acid.

Symbol Glossary:

	Manufacturer
	Use by date
	Batch number
	Catalog number
	Consult instructions for use or consult electronic instructions for use
	Caution
	Medical device
	CE Mark
	Unique Device Identifier
	Do not use if package is damaged and consult instructions for use
	Temperature limit
	Keep dry
	Keep away from Sunlight
	Keep out of reach of children

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Distributed by
3M Deutschland GmbH
Health Care Business

Carl-Schurz-Str. 1
41453 Neuss, Germany
3M.com
For more information see, HCBRegulatory.3M.com

3M and Kerrasol are trademarks of 3M.
© 2022, 3M.
Unauthorized use prohibited. All rights reserved.
Made in Germany
Issue 01 - Apr 2022
Issue date: 2022-05

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**Prontomed GmbH**